Veroval®

Veroval®

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
The screenshot displays the Veroval®medi.connect web application interface. At the top left, the logo "Veroval®medi.connect" is visible. The top right corner shows the user profile "Juraj Kováč" with identifiers "M1" and "M2", and the "HARTMANN" logo. A navigation bar includes tabs for "Prehľad", "Krvný tlak", "EKG", "Hmotnosť", "Teplota", and "Domá". The "EKG" tab is active. Below the navigation bar, there are search filters for "Týždeň" (Week) and "Mesiac vybrať" (Select month) with date pickers for "07.09.2017" and "13.09.2017", and a "Hľadať" (Search) button. A central modal window titled "Info" is open, displaying a search progress message: "Prebieha načítanie dát. Počkajte chvíľu, prosím." (Data loading is in progress. Please wait a moment, please.) Below the modal, there are "Skenuvanie" (Scan) and "Zrušiť" (Cancel) buttons. On the right side, a "Priemer" (Average) section shows a heart icon and the text "Tepová frekvencia" (Heart rate) with a dash below it. The bottom right corner features "Excel" and "Print" icons.

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The screenshot displays the Veroval® medi.connect web application interface. At the top left, the logo 'Veroval® medi.connect' is visible. The top right corner shows the user profile 'Juraj Kováč' with 'M1 M2' and the 'HARTMANN' logo. A navigation bar includes 'Prehľad', 'Krvný tlak', 'EKG', 'Hmotnosť', 'Teplota', and 'Domov'. The 'EKG' tab is active. Below the navigation bar, there are search filters for 'Týždeň' (Week) and 'Mesiac vybrať' (Select month) with date ranges '07.09.2017' and '13.09.2017', and a 'Hľadať' (Search) button. A modal window titled 'Info' is open, displaying a search result for 'Veroval ECG' with a checkmark and a refresh icon. The modal also lists 'Zoznam nájdených zariadení s ikonou a názvom', 'Zoznam spárovaných zariadení', and 'Zoznam nespárovaných zariadení'. At the bottom of the modal are 'Skenovanie' (Scanning) and 'Zrušiť' (Cancel) buttons. On the right side of the main interface, there is a 'Priemer' (Average) section with a heart icon and the text 'Tepová frekvencia' (Heart rate) and a dash '-'. Below this is an 'Excel' button and a folder icon.

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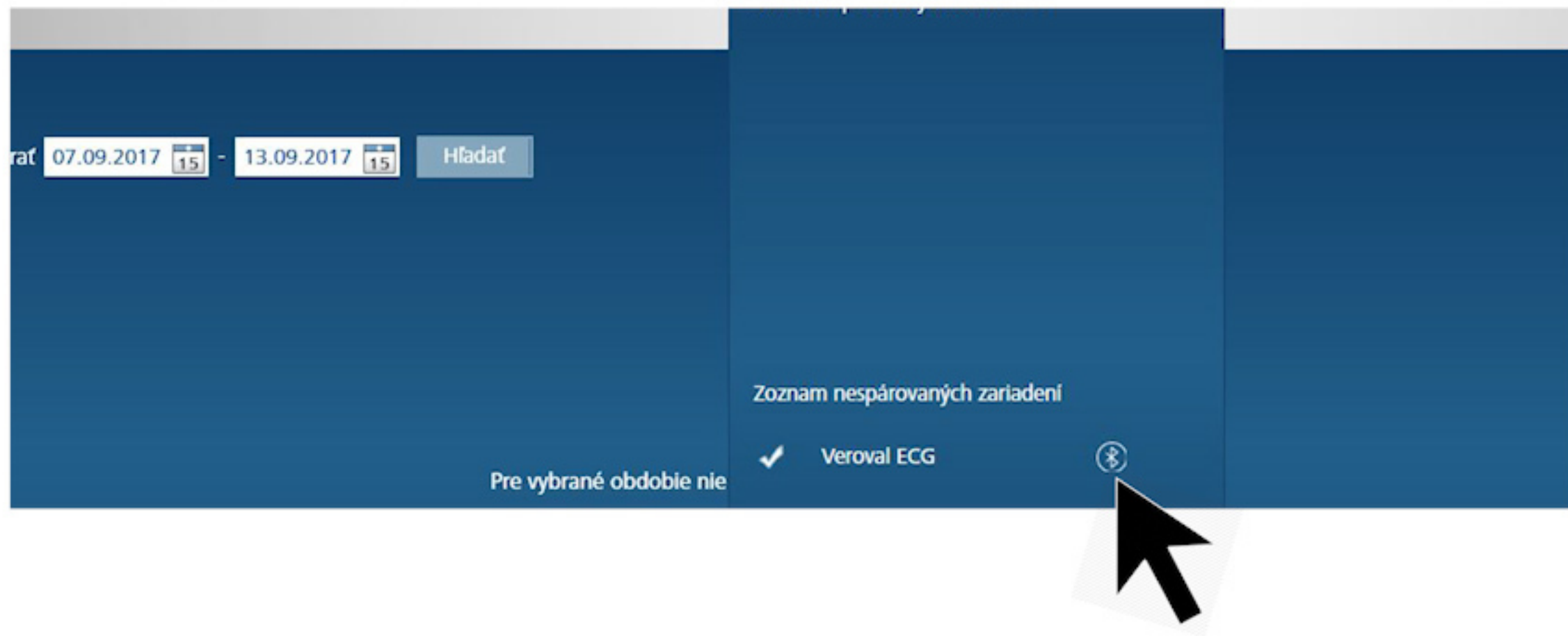Veroval®

The screenshot displays the Veroval® medi.connect web application interface. At the top, the user is identified as Juraj Kováč. The main navigation bar includes sections for 'Prehľad', 'Krvný tlak', 'EKG', 'Hmotnosť', and 'Teplota'. The 'Môj profil' (My Profile) section is active, showing personal details such as name (Juraj Kováč), date of birth (16.02.1977), and age (40). A modal window titled 'Info' is open, showing a list of devices. The 'Zoznam spárovaných zariadení' (List of paired devices) section contains one entry: 'Veroval ECG', which has a checkmark and a new device icon. A mouse cursor is pointing at this icon. Below the modal, there are buttons for 'Skenovanie' (Scanning) and 'Zrušiť' (Cancel).

Kliknite na novú ikonku na prenos údajov.

The screenshot displays the Veroval® medi.connect web application interface. At the top left, the logo "Veroval® medi.connect" is visible. The top right corner shows the user profile "Juraj Kováč" with identifiers "M1" and "M2", and the "HARTMANN" logo. A navigation menu includes "Prehľad", "Krvný tlak", "EKG", "Hmotnosť", "Teplota", and "Domá". The "EKG" tab is active, showing a date range selector for "Týždeň" (07.09.2017 to 13.09.2017) and a "Hľadať" button. A central modal window titled "Info" is open, displaying a list of devices: "Zoznam nájdených zariadení s ikonou a názvom" and "Zoznam spárovaných zariadení". Below this, a checkmark indicates "Veroval ECG" is selected. A large white box in the center of the modal contains a loading spinner and the text "Prebieha načítanie dát. Počkajte chvíľu, prosím." At the bottom of the modal are "Skenuvanie" and "Zrušiť" buttons. On the right side of the main interface, a "Priemer" (Average) section shows a heart icon and "Tepová frekvencia" (Heart rate) with a dash "-" below it. The background of the EKG section shows the text "Pre vybrané obdobie nie" (For the selected period none).

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The screenshot displays the Veroval®medi.connect web interface. At the top, the user is identified as Juraj Kováč, M1, M2. The navigation menu includes: Prehľad, Krvný tlak, EKG, Hmotnosť, Teplota, Domáci test, Aktivita, and Nastavenia. The main content area is divided into two sections:

- EKG Section:** Shows a table of EKG measurements with columns for Date, Time, Patient Name, and various metrics. A modal dialog is open, titled "Preenos dát pamäť všetkých užívateľov v prístroji" (Transfer data memory of all users in the device), with a sub-header "Zo zariadenia boli prenesené nasledujúce údaje: EKG" (From the device, the following data were transferred: EKG). The table in the dialog has columns: "Pamäť uložená v prístroji" (Memory stored in device), "Používateľ" (User), "Celkový počet prenesených meraní" (Total number of transferred measurements), "Duplicitné záznamy" (Duplicate records), "Nové merania" (New measurements), and "Detail". The data row shows: 1 measurement for Juraj Kováč, 3 total measurements, 0 duplicate records, and 3 new measurements. A "Uložiť" (Save) button is at the bottom of the dialog.
- Priemer (Average) Section:** Displays a heart rate icon and the text "Sredná frekvencia 105" (Average frequency 105). The number 105 is circled in red.

At the bottom of the interface, a red warning icon is followed by the text: "Pri zvýšených hodnotách: Navštívte, prosím, svojho lekára za účelom vyšetrenia." (In case of increased values: Please visit your doctor for examination.)

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The screenshot displays the Veroval medi.connect user interface. At the top, the user is identified as Juraj Kováč (M1, M2) with the HARTMANN logo. A navigation bar includes options like Prehľad, Krvný tlak, EKG, Hmotnosť, Teplota, Domáci test, Aktivita, and Nastavenia. The EKG section is active, showing a table of measurements for the week of 07.10.2017 to 13.10.2017. The table lists three entries with green status indicators and heart rate values of 84, 83, and 95. A 'Priemer' (Average) summary on the right shows a heart rate of 109. A footer message states: 'Hodnoty v zelenej oblasti: Kontrolujte, prosím, hodnoty naďalej pravidelne sami.'

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