Veroval[®] compact

BPW 22

ES - Tensiómetro para la muñeca BPW 22 <i>Instrucciones de uso</i>	2-24
PT – Tensiómetro de pulso BPW 22 <i>Instruções de utilização</i>	25-48
EN – BPW 22 Wrist blood pressure monitor <i>Instructions for use</i>	49-70
Documento de garantías / Certificado de garantia / Warranty certificate	71

HARTMANN

Apreciado cliente:

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Dear customer,

Thank you for choosing a HARTMANN blood pressure monitor. The Veroval® BPW 22 is a quality product for fully automated blood pressure measurement on the wrists of adults and is suitable for clinical and domestic use. Requiring no preliminary settings, this device easily and automatically inflates for simple, quick and reliable systolic and diastolic blood pressure and pulse rate measurement.

The monitor also indicates any irregular heartbeats.

We wish you the best of health.

Please read these Instructions for Use carefully before handling the device for the first time. Correctly measuring your blood pressure depends on an appropriate handling of the device. These instructions for use are designed to teach you each step in taking your own blood pressure measurements using the Veroval® BPW 22. You will be given important and useful hints to ensure that you obtain reliable results for your personal blood pressure profile. Use the monitor in accordance with the information provided in the instructions for use. Please store these instructions carefully and ensure other users can access them. Check the packaging for any damage and that the contents are complete.

Included in delivery are:

- Blood pressure monitor
- 2 x 1.5 V AAA batteries
- Storage box
- Instructions for use with warranty certificate



Table of Contents**Page**

1. Device and display description	51
2. Important information	52
3. Blood pressure information	58
4. Preparing the measurement	59
5. Measuring blood pressure	60
6. Memory function	63
7. Explanation of error messages.....	65
8. Device care.....	66
9. Warranty conditions	67
10. Contact details for customer enquiries.....	68
11. Technical information	68


1. Device and display description

Blood pressure monitor

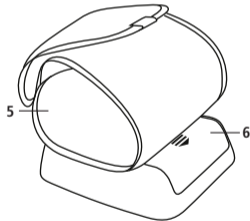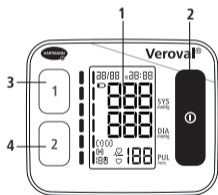- 1 Extra large LCD display
- 2 START/STOP button
- 3 Memory button for User 1
- 4 Memory button for User 2
- 5 Wrist cuff
- 6 Battery compartment

Display

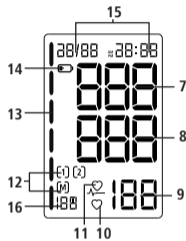- 7 Systolic blood pressure
- 8 Diastolic blood pressure
- 9 Pulse rate
- 10 Flashes when the device is measuring and the pulse rate is detected
- 11 Irregular heartbeat
- 12 User memory
- 13 Readings shown in a traffic light format
- 14 Battery symbol
- 15 Date and time display
- 16 Average value (A), morning (AM), evening (PM) / Memory space number

2. Important information

Signs and symbols

The following symbols are used in the Instructions for Use, on the packaging and on the type plate of the device and the accessories:

Consulting the Instructions for Use

Please note

IP22

Protected against penetration by solid objects with a diameter > 12.5 mm.

Protected against water droplets falling on the monitor at an angle of up to 15°.

Temperature limits

Air humidity limits

Protection from electric shock

Dispose of packaging in an environmentally responsible way

Dispose of packaging in an environmentally responsible way

Recycle: Electronic equipment/Con marking electrical and electronic equipment

Labelling in accordance with Medical Devices Directive 93/42/EEC

Direct current

Manufacturer

Authorised representative in the European Community

LOT

Batch code

REF

Catalogue number

Cardboard recycling code

SN

Serial number

**Important instructions for use**

- Only use the monitor to take blood pressure measurements on the human wrist. Do not position the cuff on any other areas of the body.
- Only use the monitor on persons whose wrist circumference is within the range indicated for the monitor.
- If measured values seem questionable, repeat the measurement.
- Never leave the device unattended near small children or persons who cannot operate it themselves. Swallowing small parts that have detached from the monitor can cause suffocation.
- Never carry out blood pressure measurements on newborn infants, babies or toddlers.
- Please do not place the cuff on a wound as this may cause further injuries.
- Do not place the cuff on persons who have had a mastectomy.
- Please note that the pressure build-up in the cuff can temporarily disrupt medical devices being used on that arm at the same time.
- Do not use the blood pressure monitor in conjunction with a high-frequency surgical device.
- If an intravenous treatment is being carried out or a venous catheter has been inserted in the arm, the blood pressure measurement process can result in injury. Never use the cuff on the arm to which these conditions apply.
- The function of the arm on which the cuff is placed may become impaired during inflation.
- If you are taking another person's blood pressure, please ensure that using the blood pressure monitor does not have a lasting negative effect on blood circulation.
- Taking too many measurements within a short period of time and maintaining continuous cuff pressure can disrupt blood circulation and cause injury. Please take a break between measurements. If the monitor malfunctions, remove the cuff from the wrist.

- Do not use the blood pressure monitor on patients with pre-eclampsia during pregnancy.

Important instructions on taking own blood pressure

- Even minor changes in internal and external factors (such as deep breathing, stimulants, talking, excitement, environmental factors) can cause fluctuations in blood pressure. This explains why anomalous values are often measured by a doctor or pharmacist.
- Measurement results fundamentally depend on the measurement location and the position of the patient (sitting, standing, reclining). They are also influenced by e.g. physical activity and physiological preconditions of the patient. For comparable values, carry out the measurement in the same location and in the same position.
- Cardiovascular disease can result in erroneous readings or lower the accuracy of the measurement. The accuracy may also be affected if you have very low blood pressure, diabetes, circulatory disorders, arrhythmias, and chills or tremors.

Consult your doctor before taking your own blood pressure readings if ...

- you are pregnant. Blood pressure may vary during pregnancy. Regular blood pressure monitoring is particularly important if you have high blood pressure, because elevated values may affect foetal development. Always check with your doctor if and when you should measure your own blood pressure, especially if you suffer from pre-eclampsia.
- you have diabetes, liver function disorders or narrowing of blood vessels (e.g. atherosclerosis, peripheral arterial disease), as these may cause anomalous values.
- you have certain blood disorders (e.g. haemophilia) or severe circulatory disorders or are taking blood-thinning medication.
- you wear a pacemaker, which may cause anomalous values. The blood pressure monitor itself does not affect a pacemaker. Please note that the pulse rate displayed is not suitable for checking the frequency of pacemakers.
- you tend to bruise easily and/or are sensitive to pressure pain.

- you suffer from severe heart rhythm disorders or cardiac arrhythmias. Due to the oscillometric measurement method used, incorrect measured values may be obtained in some cases or it may not be possible to measure blood pressure.
- If this symbol  appears often, it may indicate heart rhythm disorders. Please consult your doctor in that case. Severe heart rhythm disorders may sometimes produce lead to measurements or reduce measurement accuracy. Please discuss with your doctor whether taking your own blood pressure is appropriate for you.
- The readings taken by you are for your information only – they are not a substitute for a medical examination. Discuss your measured values with your doctor and never make a medical decision based on these measurements (e.g. medication and dosage).
- Taking your own blood pressure is not a substitute for medical treatment. Do not interpret your measured values yourself and do not use them to self-prescribe treatment. Take readings based on the instructions of your doctor and trust his or her diagnosis. Take medication only as prescribed by your doctor and do not alter the dose on your own accord. Discuss with your doctor the appropriate times to measure your blood pressure.

Notes on handling batteries

- Observe the polarity labels plus (+) and minus (-).
- Use only premium quality batteries (see specifications in Chapter 11 - Technical Information). We cannot guarantee the stated measurement performance if low-quality batteries are used.
- Never mix old and new batteries or batteries from different manufacturers.
- Remove empty batteries immediately.
- Replace batteries if the battery symbol  remains illuminated.
- Always replace all batteries at the same time.
- If you do not intend to use the monitor for a longer period of time, you should remove the batteries to prevent possible leakage.

Notes on batteries

- **Choking hazard**
Small children could swallow batteries and suffocate on them. Keep batteries out of the reach of children.
- **Risk of explosion**
Do not throw batteries into a fire.
- Batteries must not be charged or short-circuited.

- If a battery has leaked, wear protective gloves and clean the battery compartment with a dry cloth. If liquid from a battery cell comes into contact with skin or eyes, clean the affected area with water, and seek medical attention if necessary.
- Protect batteries from excessive heat.
- Do not disassemble, open or crush batteries.

Safety information concerning the device

- This blood pressure monitor is not waterproof.
- This blood pressure monitor is made of high-quality electronic precision components. The accuracy of the measured values and the lifetime of the device depend on careful handling.
- Protect the device from violent shaking, hitting or vibrations and do not let it drop on the ground.
- Do not bend or fold the cuff excessively.
- Never open the device. The device must not be modified or dismantled, and must not be repaired by the user. Repairs may only be carried out by an authorised specialist.
- Inflate the cuff only after it has been positioned properly around the wrist.

- Do not expose the monitor to extreme temperatures, humidity, dust or direct sunlight as this may cause it to malfunction.
- Keep the packaging, batteries and the device out of the reach of children.
- Please comply with the storage and operating conditions defined in Chapter 11 - Technical Information. Storing or using the device outside of the specified temperature and humidity range can affect measurement accuracy as well as the function of the device.
- Do not operate the device near strong electromagnetic fields and keep it away from radio equipment and mobile phones. Portable and mobile high-frequency and communication devices, such as telephones and mobile phones, can impair the functionality of this electronic medical device.

Notes on electromagnetic compatibility

- The device is suitable for use in all environments listed in these Instructions for Use, including domestic environments.
- It may only be possible to use the device to a limited extent in the presence of electromagnetic disturbances. This could result in issues such as error messages or the failure of the display/device.
- Avoid using this device directly beside other devices or stacked on top

of other devices, as this could result in faulty operation. However, if it is necessary to use the device in the manner described above, this device and the other devices must be monitored to ensure that they are working properly.

- The use of accessories other than those specified or provided by the manufacturer of this device may result in increased electromagnetic emissions or a decrease in the device's electromagnetic immunity and lead to faulty operation.
- Failure to comply with the above can impair the performance characteristics of the device.

Notes on checking calibration

Every Veroyal® device has been carefully tested by HARTMANN for measurement accuracy and has been developed with a view to a long service life. We recommend carrying out a calibration check every 2 years for devices in professional use at, for example, pharmacies, medical practices or hospitals. In addition, please also observe national regulations established by the legislator. The calibration check should only be carried out by competent authorities or authorised maintenance providers at the user's expense.

Notes on disposal

- To protect the environment, empty batteries must not be disposed of in household waste. Please comply with the relevant waste disposal regulations or use public collection points.
- This product is subject to European Directive 2012/19/EU on Waste Electrical and Electronic Equipment (WEEE) and is marked accordingly. Never dispose of electronic equipment in your household waste. Please consult the local regulations regarding the proper disposal of electrical and electronic products in your area. Proper disposal protects the environment and public health.

3. Blood pressure information

Two values must be measured to determine blood pressure:

- Systolic (upper) blood pressure, which is produced when the heart contracts and pumps blood into the blood vessels.
- Diastolic (lower) blood pressure, which is produced when the heart is expanded and fills with blood again.

■ Blood pressure values are displayed in mmHg.
For improved assessment of the results, a traffic-light system is located on the left side of the Veroval® BPW 22 which is a direct result indicator, making it easier to categorise the measured value. The World Health Organization (WHO) and the International Society of Hypertension (ISH) have developed the following outline for classifying blood pressure values:

Result indicator	Evaluation	Systolic pressure	Diastolic pressure	Recommendation
red	Stage 3 hypertension	above 179 mmHg	above 109 mmHg	Consult a doctor
orange	Stage 2 hypertension	160 - 179 mmHg	100 - 109 mmHg	
yellow	Stage 1 hypertension	140 - 159 mmHg	90 - 99 mmHg	Regular examinations by a doctor
green	Normal limit values	130 - 139 mmHg	85 - 89 mmHg	
green	Normal	120 - 129 mmHg	80 - 84 mmHg	Self-assessment
green	optimal	up to 119 mmHg	up to 79 mmHg	

Source: WHO, 1999 (World Health Organization)

4. Preparing the measurement

Positioning / changing the batteries


Fig. 1

- Open the battery cover on the bottom of the device (see Fig. 1). Insert the batteries (see Chapter 11 - Technical Information). Ensure correct polarity (+ and -) when inserting batteries. Close the battery lid. 12 h or 24 h flashes on the display. Set date and time as described below.
- If the 'Change battery' symbol  is permanently illuminated, blood pressure can no longer be measured and you need to replace all of the batteries.

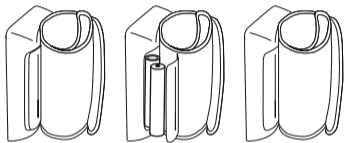Setting the date and time



Be sure to set the date and time correctly. This is the only way to save your measured values correctly with date and time for subsequent retrieval.

- To switch to Settings, reinsert the batteries or press the START/STOP button  for 5 seconds. Proceed as follows:

Hours

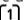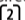The hour format flashes on the display.

- Select the desired hour format using the Save buttons  /  and confirm using the START / STOP button .

Date

The display shows the year (a), the month (b) and the day (c).

- Depending on the display, use the Save buttons [1] / [2] to select the year, the month or the day and confirm using the START / STOP button [I].

i If the 12-hour format is selected, the month is displayed before the day.

Time

The hour (d) and the minutes (e) flash in sequence on the display.

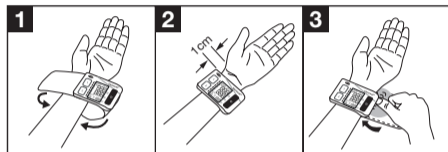
- Depending on the display, use the Save buttons [1] / [2] to select the current hour or minutes and confirm using the START / STOP button [I].

The device switches off automatically once all data have been set.

5. Measuring blood pressure

Applying the blood pressure monitor

- Blood pressure must be measured on a bare wrist. The cuff should not be positioned over protruding wrist bones as it will not fit evenly around the wrist.
- The monitor has a fixed connection to the cuff and the cuff must not be detached from the monitor.
- Now pull the cuff over the wrist. The blood pressure monitor is positioned centrally on the inside of the wrist approx. 1-1.5 cm from the crease (carpus) of wrist.

The cuff should fit firmly, but not tightly. Please note that incorrect positioning of the cuff may produce an incorrect measurement. Use the Veroval® BPW 22 only with the specially approved cuff supplied. It has a cuff circumference of 12.5 - 21 cm. Using a Veroval® upper arm blood pressure monitor is recommended for wrist circumferences larger than that, because the accuracy of the blood pressure readings cannot be guaranteed otherwise.

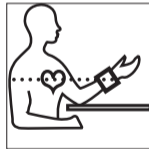
This innovative Veroval® device with Comfort Air Technology ensures comfortable measurement. Inflation to 190 mmHg is required for the first measurement. For subsequent measurements, inflation pressure is adjusted individually based on the blood pressure values recorded previously. This allows for more comfortable measurements.

Taking the measurement

- Rest for approx. 5 minutes before taking the measurement.
- You should take your blood pressure in a quiet place, in a relaxed and comfortable seated position.

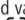
- Measurement can be taken on the right or left wrist. We recommend performing the measurement on the left wrist. Over the long term, the wrist giving higher readings should be used for blood pressure monitoring. However, if there is a very clear difference between readings on either wrist, you need to check with your doctor which wrist you should use for the measurement.
- Always take measurements on the same wrist.
- Take measurements on a bare wrist and while sitting upright.
- We recommend that you measure your blood pressure while sitting with your back against the back of the chair. Place both feet flat on the floor next to each other. Do not cross your legs. Be sure to support your arm and bend it. Always ensure that the cuff is positioned at the level of the heart. If not, measurements may vary considerably. Relax your arm and the palm of your hand.
- Do not measure your blood pressure after taking a bath or playing sport.

- Do not eat, drink or exercise for at least 30 minutes before the measurement.
- Please wait for at least one minute between two measurements.
- Always take measurements at the same time of day. Only regular measurements taken at the same time each day over an extended period give a meaningful assessment of blood pressure values.
- Do not measure when you have a strong urge to urinate. A full bladder can increase blood pressure by about 10 mmHg.
- Do not start a measurement until you have put the device on. Press the START/STOP button . The appearance of all display segments, followed by time and date, indicates that the device is carrying out an automated check and is ready for use.
- Check the display segments for completeness (see Chapter 1).
- The cuff automatically inflates after about 0.5 seconds. If this inflation pressure is insufficient or if the measurement is interrupted, the device continues to pump in increments of 40 mmHg until a high enough pressure is reached. During inflation, the result display on the left also increases at the same time.

Important: Do not speak or move for the duration of the measurement.

- As the cuff pressure is released, the heart symbol  flashes and the drop in cuff pressure is shown on the display.
- After the measurement is complete, the systolic and diastolic blood pressure as well as the pulse rate (see Fig. 1) are shown simultaneously.
- In addition to the measured values, the time, date, associated user memory or  and associated memory numbers (e.g. ) are shown. The measured value is automatically stored in the displayed user memory. While the result of the measurement is displayed, you can assign the values to the corresponding user memory by pressing the  or  button. If you do not assign them, the measured values are automatically stored in the user memory shown on the display. You can use the result indicator on the left in the display to classify your measurement result (see table in Chapter 3 - Blood Pressure Information).
- To switch off the device, press the START/STOP button , otherwise the device switches off automatically after 60 seconds.

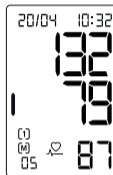
Fig. 1

If you wish to stop the measurement for any reason, simply press the START/STOP button **(I)**. The inflation or measurement process is interrupted and cuff pressure is automatically released.

- If this symbol  is shown in the display, the device has detected an irregular heartbeat during the measurement. However, the measurement may also have been interrupted by body movement or speaking. It is best to repeat the measurement. If you see this symbol regularly when measuring your blood pressure, we recommend that your doctor checks your heart rhythm.

6. Memory function

User memory

- The Veroval® BPW 22 stores up to 100 measurements in each user memory. Once the memory is full, the oldest value is deleted.
- Memory recall is activated by pressing the **(1)** or **(2)** button when the device is switched off. For values saved in the first user memory, press the **(1)** button, for the second user memory press the **(2)** button.

Average values

- After selecting the respective user memory, the corresponding symbol **(1)** or **(2)** and an A appear on the display. The average value of all the data stored for that particular user memory is displayed (see Fig. 1).


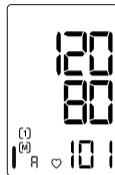
Fig. 1

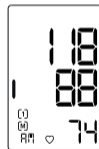- By pressing the **(1)** button again (or the **(2)** button if you are in user memory 2), the average values for all morning measurements 'AM' (5 am to 9 am) taken over the last 7 days are shown.

- By pressing the **[1]** button again (or the **[2]** button if you are in user memory 2) the average values for all evening measurements 'PM' (6.00 pm to 8.00 pm) taken over the last 7 days are shown.

Individual measured values

- By pressing the **[1]** button again (or the **[2]** button, if you are in user memory 2), all memory values can be retrieved one after the other, starting with the most recently measured value.

- If an irregular heartbeat was detected during measurement, that information  is also stored and displayed when the measured value is retrieved from the device's memory, along with the systolic and diastolic blood pressure readings, pulse rate, time, date and year.
- You can cancel memory recall at any time by pressing the START/STOP button **[I]**. The device switches off automatically otherwise after a few seconds.
- The stored values will still be available even if the power supply is interrupted, e.g. when changing the batteries.

Deleting stored readings

All data saved for each user can be deleted separately for user memory **[1]** and user memory **[2]**. To do this, press the key of the corresponding user memory (**[1]** or **[2]**). The average value is then shown on the display.

Press and hold the user memory button for 5 seconds. 'CL 00' appears on the display. All data in the selected user memory have now been deleted. If you release the button early, no data will be deleted.

7. Explanation of error messages

Error that occurred	Possible causes	Remedy
Monitor cannot be switched on	Batteries are missing, positioned incorrectly or flat.	Check batteries. If necessary, replace with two new identical batteries.
E1	The measuring signals could not be detected at all or read correctly. This can be caused by applying the cuff incorrectly, by moving or talking, or by a very weak pulse.	Check that the cuff has been positioned correctly. Do not talk or move during measurement.
E2	Incorrect measurement due to movement.	Do not talk or move during measurement.
E3	The cuff was not applied correctly.	Check that the cuff has been positioned correctly.
E4	Error during measurement.	Contact customer service if this error message appears.
E5	Inflation pressure is higher than 300 mmHg.	Please rest for at least 1 minute, then take the measurement again.
E6	A system error has occurred.	Contact customer service if this error message appears.

Error that occurred	Possible causes	Remedy
	The batteries are almost flat.	Replace the batteries.
Implausible measurements	Implausible measurements often occur if the device is handled incorrectly or if mistakes were made during the measurement process.	Please refer to Chapter 5 - Measuring Blood Pressure, as well as the Notes on Safety. Then repeat the measurement.

Switch the device off if an error message appears. Check for possible causes as well as the notes on performing the measurement yourself in Chapter 2 - Important Notes. Relax for at least 1 minute and then take the measurement again.

8. Device care

- Only use a soft, damp cloth to clean the device. Please do not use thinners, alcohol, detergents or solvents.
- The cuff can be cleaned carefully using a slightly damp cloth and a mild soapy solution. Do not immerse the cuff in water completely.
- Cleaning and disinfecting the cuff regularly or after each use is recommended to prevent any infections, especially if it is used by multiple persons. The cuff should be disinfected, particularly on the inside, by wiping with a disinfectant. Use a disinfectant that is compatible with the cuff materials, e.g. 75% ethanol or isopropyl

alcohol. Keep the device in the storage box to protect it from external influences.

- In storage, do not place any heavy objects on top of the device and the cuff. Remove the batteries.

9. Warranty conditions

- We offer a 3-year warranty on this high-quality blood pressure monitor from the date of purchase and in accordance with the conditions listed below.
 - Claims must be made within the warranty period. The date of purchase must be documented by an appropriately completed and stamped warranty certificate or proof of purchase.
 - HARTMANN will replace or repair the device free of charge in the event of any material and manufacturing defects within the warranty period. This shall not extend the warranty period.
 - The device is intended only for the purpose described in this instruction manual.
 - The warranty does not cover damage caused by improper handling or tampering. Accessory parts that are subject to wear and tear (batteries, cuffs, etc.) are excluded from the warranty. Claims for compensation are limited to the value of the merchandise; compensation for consequential damages is expressly excluded.
- In the event of a warranty claim, please send the device with cuff and the fully completed and stamped warranty certificate or proof of purchase directly or via your dealer to the Customer Service contact in your country.

10. Contact details for customer enquiries

DE PAUL HARTMANN AG
Service Centre Diagnostics
Friedrich-Penseler-Strasse 17
21337 Lueneburg
customer.care.center@hartmann.info
www.veroval.de

0800-400 400 9 (freephone within Germany)
Monday – Friday from 8.00 am to 4.30 pm

AT UTS Geräte Service Ges.m.b.H.
Phone: 0043 1 706 14 15
Email: office@uts.at
www.veroval.at

CH IVF HARTMANN AG
8212 Neuhausen
www.veroval.ch

11. Technical information

Model:	Veroval® BPW 22
Type:	BPM 27
Measuring method:	oscillometric
Display range:	0 - 300 mmHg
Measuring range:	Systolic (SYS): 50 - 280 mmHg Diastolic (DIA): 30 - 200 mmHg Pulse: 40 - 199 pulse beats per minute We cannot guarantee that values outside the measuring range will be displayed correctly.
Display unit:	1 mmHg
Technical measurement accuracy:	Cuff pressure: ± 3 mmHg Pulse: $\pm 4\%$ of the displayed pulse rate

Clinical measuring accuracy:	Complies with the requirements of EN 1060-4; Korotkoff validation method: Phase I (SYS), Phase V (DIA)
Operating mode:	Continuous mode
Nominal voltage:	DC 3V
Expected service life:	10,000 measurements
Power supply:	2 x 1.5 V alkaline manganese (AAA/LR03) batteries
Battery capacity:	Approx. 1,000 measurements
Protection against electric shock:	Internally powered ME device Application part: Type BF
Protection against harmful ingress of water or solid materials:	IP22 (not waterproof)
Inflation pressure:	About 190 mmHg for first measurement

Automatic switch-off:	1 minute after measurement / otherwise 30 sec.
Cuff:	For wrist diameters of 12.5 - 21 cm
Memory capacity:	2 x 100 measurements with average value of all measurements and morning / evening average value of the last 7 days
Operating conditions:	Ambient temperature: +10°C to +40°C Relative humidity: ≤ 85%, non-condensing Air pressure: 800 - 1050 hPa
Storage / transport conditions:	Ambient temperature: -20°C to +50°C Relative humidity: ≤ 85%, non-condensing
Serial number:	Inside the battery compartment

Reference to standards:	IEC 60601-1; IEC 60601-1-2 (in compliance with CISPR 11, IEC 61000-4-2, IEC 61000-4-3, IEC 61000-4-8)
Weight:	approx. 115 g (without batteries)
Dimensions:	approx. 70(L) x 85(W) x 24(H) mm

Legal requirements and guidelines

- The Veroval® BPW 22 conforms to the European regulations that are based on the European Medical Device Directive 93/42/EEC, and bears the CE mark.
- The device complies, for example, with the European standard EN 1060: Non-invasive Sphygmomanometers – Part 3: Additional requirements for electro-mechanical blood pressure measuring systems and the standard IEC 80601-2-30.
- Clinical testing of measuring accuracy was performed in accordance with the European Standard EN 1060-4.
- Beyond the statutory requirements, the device has been clinically validated by the ESH-IP2 Protocol of the European Society of Hypertension (ESH).

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Sello del vendedor · Carimbo do vendedor · Dealer's stamp

Date of revision: 2019-02

Made under the control of PAUL HARTMANN AG

AViTA Corporation
9F, No.78, Sec.1, Kwang-Fu Rd.
San-Chung District
24158 New Taipei City
Taiwan

Wellkang Ltd.
29 Harley St.
W1G 9QR
LONDON, U.K.

749.464-02-2019