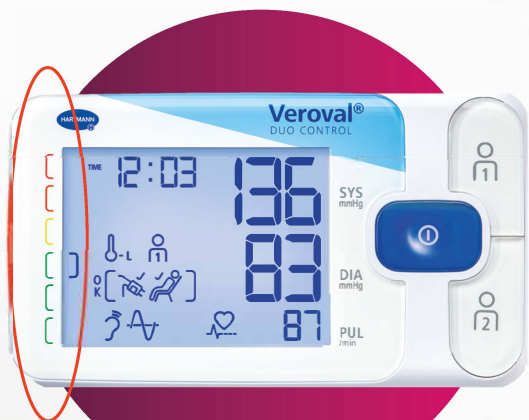
Veroval® DUO CONTROL

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NEW


Traffic light system for results

For easily understanding of the measurements, a traffic light system is available on the left side of the Veroval® devices.


Use the table to check where your measured value is categorized.

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Results indicator	Evaluation	Systolic pressure	Diastolic pressure	Recommendation
Red	Grade 3 hypertension	above 179 mmHg	above 109 mmHg	Consult a doctor
Orange	Grade 2 hypertension	160 - 179 mmHg	100 - 109 mmHg	
Yellow	Grade 1 hypertension	140 - 159 mmHg	90 - 99 mmHg	Regular examinations by a doctor
Green	Normal limit values	130 - 139 mmHg	85 - 89 mmHg	
Green	Normal	120 - 129 mmHg	80 - 84 mmHg	Self-assessment
Green	Optimal	up to 119 mmHg	up to 79 mmHg	

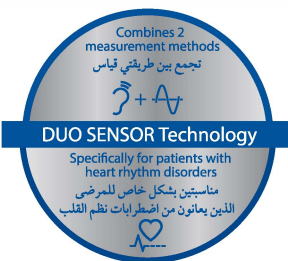Type	Size of cuff	Article No.
Veroval® Duo Control medium	22 - 32 cm	925 506
Veroval® Duo Control large	32 - 42 cm	925 526
Cuff for Veroval® Duo Control medium	22 - 32 cm	925 531
Cuff for Veroval® Duo Control large	32 - 42 cm	925 532

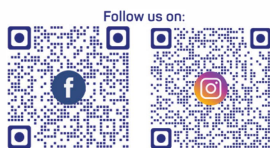PAUL HARTMANN Middle East
Dubai Airport Freezone
P.O. Box 54525 Dubai
United Arab Emirates
Phone: +971 4 2996996

me.hartmann.info

Accuracy even with heart rhythm disorders

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Easy data administration and saving

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For hypertensive patients and those who suffer from cardiac arrhythmias and have to measure regularly.

- ✓ Medically precise measurements using the patented **DUO SENSOR** technology
- ✓ Fully automatic blood pressure and pulse measurement on the upper arm
- ✓ Easier correct measurement thanks to the patented Secure Fit cuff, cuff fit check and resting indicator
- ✓ Detects arrhythmia and measures correctly
- ✓ Large display with backlight
- ✓ Traffic-light system for your readings
- ✓ Averages of all stored measured values
- ✓ Averages of morning and evening blood pressure values from past 7 days
- ✓ Memory for 200 readings (100 readings per user, max. 2 users)
- ✓ Developed in Germany

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Keep track of your values using **Veroval®** medi.connect

USB connection
(USB cable included)

Free software download or web access:
www.veroval.info

The 10 Golden Rules for correct measurement of blood pressure

Veroval®
BLOOD PRESSURE | ضغط الدم

1. Rest for about 5 minutes before taking a measurement. Desk work increases blood pressure on average by about 6 mmHg systolic and 5 mmHg diastolic.

2. Do not smoke or consume coffee up to one hour before the measurement.

3. Do not measure your blood pressure if you urgently need to urinate. A full bladder can lead to an increase in the blood pressure of about 10 mmHg.

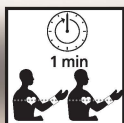
4. Measure your blood pressure on a completely bare upper arm and in a comfortable upright seated position. The blood circulation must not be constricted by a rolled up sleeve, for example.

5. If a wrist monitor is used, hold the cuff at heart height during the measurement. The cuff of an upper arm monitor is naturally positioned at the correct height.


6. Please do not talk or move during the measuring procedure. Talking increases the values by about 6-7 mmHg.

7. Wait at least one minute between two measurements, so that the vessels are relieved from pressure in preparation for a new measurement.

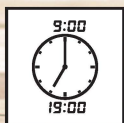8. Take measurements regularly. Even if your values have improved, you should continue to check them for monitoring purposes.

9. Always take measurements at the same time of day. A person has about 100,000 different blood pressure values during the day. Regular measurements taken at the same time each day over a longer period of time give a meaningful assessment of blood pressure values.

10. Document all measurements with date, time and medications taken, conveniently and simply with Veroval® medi.connect.

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