The interpretation of the body temperature depends on the place of measurement.

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Veroval®
FEVER | حرارة

Type	Article No.
Veroval® Fever (ear and forehead)	925 466

Type	Quantity	Article No.
Veroval® Fever caps	20 pcs	925 483

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What you should know as a user.

Facts on ear and forehead thermometers.

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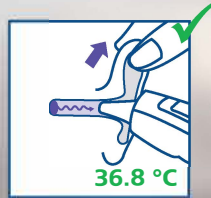
Ear thermometers measure the temperature of the eardrum, since the eardrum has nearly the same temperature as the core of our body.

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

This is why part of the procedure in taking ear measurement is to pull the ear up and backwards for adults while the outer ear to the back only for children below the age of 1.

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Forehead thermometers measure the temperature of the body via the arteria temporalis; the artery behind the forehead which emits nearly the same temperature as the core of our body.

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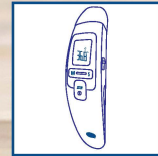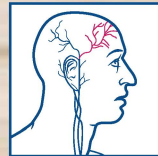The 'As-you-like-it' thermometer!

Two ways to clinically accurate results.

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Ear Measurement

Take off the forehead cap Press SCAN to switch on the device.

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Switch the mode button on the side to Ear and ensure the icon is displayed.

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Place a new, clean protective cap on the tip of the thermometer's sensor until the protective cap icon is shown on the display.

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Since the ear canal is curved, pull the ear up and backwards so that the tip can be pointed directly at the eardrum.

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*for children under the age of 1, pull the outer ear to the back only.

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Carefully insert the sensor tip and press the SCAN key for one second. The '°C' icon flashes on the display while taking the temperature.

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After about 1 second, the measurement result appears and the backlighting illuminates for 5 seconds.

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Switch the mode button on the side to Forehead and ensure the icon is displayed.

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Hold the SCAN key down and then slowly and evenly move the thermometer over the forehead to the other temple, then release the SCAN key.

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The '°C' icon flashes on the display while taking the temperature.

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The temperature appears and the background illumination lights up for 5 seconds.

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If the temperature taken is less than 37,6°C, the fever indicator lights up in green, if it is 37,6°C or higher, the fever indicator lights up in red.

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If you do not press any key for one minute, the thermometer switches off automatically.

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- ✓ 2 in 1 infrared in ear and forehead thermometer
- ✓ Fast reading in 1 second
- ✓ Hygienic thanks to single use protection caps (10 incl.)

NEW

Golden Rules for fever measurement in the ear and on the forehead


Veroval[®]
FEVER | حرارة

HARTMANN
+


1. Prior to every measurement, the thermometer and person to be measured should be at a room temperature area for at least 30 minutes.

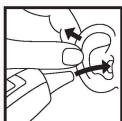Note: do not store device in an area exposed to direct heat/ sunlight (e.g car compartment, kitchen, garage)

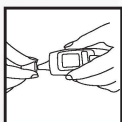2. Always use the same ear for measurement. Temperature values may vary from ear to ear.

3. Replace the single-use protective cap after each use when taking ear temperature measurements to ensure a precise measurement and to prevent cross-contamination. A single-use protective cap is not needed for forehead temperature measurement.

Only use HARTMANN single-use protective caps for replacement. Using other brands can cause errors in results.

4. For forehead application, scan over the same area every time as otherwise the temperature values may vary.

5. Wait at least for 30 seconds between two measurements.

6. Do not measure temperature directly after physical work, taking a bath, or when the ears or the forehead are dirty, sweaty or wet.

7. The measurement must not be carried out in an ear or on the forehead which exhibits inflammatory diseases, rashes and injuries.

8. Do not measure temperature if you have rested on an ear or on the forehead.

Note: Prolonged usage of earphones/ headsets prior to ear measurement can also affect the accuracy of measurement.

9. The device should be free from any residual dirt or scratches prior to usage to get accurate measurement results. Keep the lens of the sensor intact at all times. If necessary, clean the lens carefully with cotton buds. Wait 15 min if an alcohol pad is used when cleaning.

Please read the „Instruction Manual“ included with the device carefully before first use. Correct temperature measurement depends only on the appropriate application and use of the device. Those instructions describe the individual steps of temperature measurement using Thermoval duo scan and contain important and helpful hints for the reliable determination of the body temperature. Be sure to keep the manual for future reference.

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